





Caution — consult accompanying documentation.


## To Record an FVC Test

1. Connect spirometer to electrocardiograph.
2. Press .
3. Enter or search for patient data (age, height, gender, race).
4. Select effort and curve types.
5. Press **Start**. Perform effort. Press **Stop**.
6. Accept or reject effort. Repeat from Step 4 until finished.
7. Press **Test Done**.

## To Calibrate the Spirometer

1. Connect spirometer to electrocardiograph.
2. Press Menu key .
3. Choose **Spirometry Settings > Calibration Settings > Calibrate Spirometer**.
4. Enter transducer lot code and calibration code.
5. Update ambient temperature, humidity, and barometric pressure.  
(See your local weather reports.)
6. Follow prompts.

## To Configure the Spirometer

1. Connect spirometer to electrocardiograph.
2. Press Menu key .
3. Choose **Spirometry Settings**.
4. Change desired settings.