

# Welch Allyn® Ear Wash System

## Quick Reference Guide

### Initial Installation

**1** Remove the faucet aerator.

A close-up photograph of a hand using a small tool to pry a white faucet aerator from a chrome faucet. A white arrow points downwards from the aerator, indicating the direction of removal.

**2** Remove the paper tab from the snap aerator and add the Ear Wash System snap aerator (adapter if needed).

A close-up photograph of a hand holding a clear plastic snap aerator. A white arrow points upwards towards the aerator, indicating the direction of attachment to the faucet.

**3** Push the black hose into the black port and push the blue hose into the blue port until snug.

A photograph of the Welch Allyn Ear Wash System chamber. A hand is shown pushing a black hose into the black port and a blue hose into the blue port. The hoses are being pushed until they are snug.

### Cleaning Procedure

**1** Carefully grasp the pinna and administer the ear wax softener. (not required)

A close-up photograph of a person's ear. A hand is shown carefully grasping the pinna and administering a white ear wax softener into the ear canal.


**2** Attach the chamber to the sink by pushing down on the white ring at the top of the chamber and snapping onto faucet.

A photograph showing the Welch Allyn Ear Wash System chamber being attached to a faucet. A hand is shown pushing down on the white ring at the top of the chamber and snapping it onto the faucet.


**3** Turn the hot and cold water on full power. Adjust by turning the cold down until the water is warm. TIP: Hotter water is better.

A photograph of the Welch Allyn Ear Wash System chamber installed on a sink faucet. Water is being dispensed from the chamber into a white cup.


**4** Attach the ear tip to the handle. Verify the tab is on the top.




**5** Squeeze the Actuator (A) and begin irrigating the ear when the blue temperature sensor (B) changes to white.

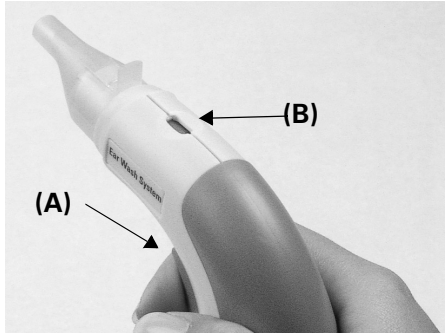

**7** Pull tab to remove the ear tip from the handle and dispose of properly.

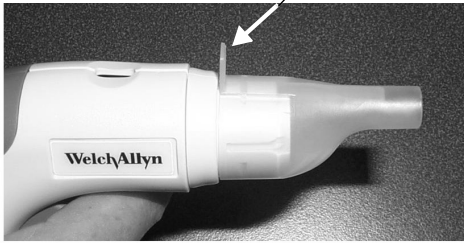


**6** Rotating the handle during the procedure while holding the pinna encourages wax removal.



Always inspect the ear before attempting any irrigation. If perforation is suspected and the tympanic membrane is not visible, do not irrigate with this device.

Refer to page 2 of Operating Instruction Manual for further information.

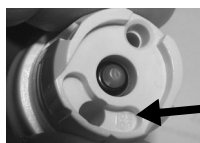

The water chamber is designed for use ONLY with handle dated AFTER

02 (YEAR)


03 (MONTH)



**For Safe Use**

Plastic flow limiter must be in place for all aerators.

See page 4 in manual.

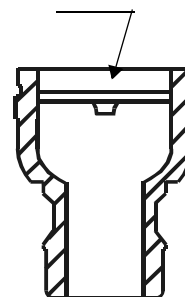

Visit our website at [www.earwash.welchallyn.com](http://www.earwash.welchallyn.com) for more information.