

Atlas Monitor Competency Checklist

Name: _____ Unit: _____ Date: _____

- _____ Demonstrate how to turn on the Atlas Monitor. Now locate the AC indicator –is the battery is being charged?
- _____ True or False – When the monitor is turned on, all alarms are enabled at the default limits and there is no trend data.
- _____ Obtain a manual NIBP from the Atlas monitor
- _____ Change the NIBP to automatic mode; then change the time intervals
- _____ Demonstrate how to SUSPEND the blood pressure alarms
- _____ Demonstrate how to SILENCE an alarm. How long will they be silenced? What is the default?
- _____ Demonstrate how to change alarm limits for each monitored vital sign
- _____ Identify the Alarm Volume button and adjust the sound
- _____ Demonstrate how to adjust the SPO2 pulse tone volume. Can it be turned to off?
- _____ Demonstrate how to change ECG lead display. Can you use 3 or 5 lead cables? If so, how?
- _____ Which ECG lead is the respiration waveform derived from?
- _____ Display and print the Trend data; how many trends can be stored in the Atlas?
- _____ Display and print Waveforms. How many seconds of data will be printed?
- _____ True or False – The Atlas monitor should be turned off between patients.