



## Connex Vital Signs Monitor 6000 Series (CVSM) Competency Checklist

Name: \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

- \_\_\_\_\_ Locate the Power button on the CVSM 6000 series and turn on the device.
- \_\_\_\_\_ The CVSM 6000 device is intended to take vital sign measurements on Adult, Pediatric, and Neonate patients. Demonstrate how to change modes.
- \_\_\_\_\_ Obtain a manual NIBP from the CVSM 6000 device; then cancel a blood pressure in progress. Change the NIBP to intervals; adjust settings to 5-minute intervals.
- \_\_\_\_\_ (if applicable) Demonstrate how to adjust high/low alarm settings for pulse rate.
- \_\_\_\_\_ (if applicable) Obtain an oral temperature. How will probe location impact the results? Now change settings to an Adult Axillary.
- \_\_\_\_\_ Demonstrate two ways to SUSPEND an audible alarm from the Home screen.
- \_\_\_\_\_ Demonstrate how to Review past vital signs for your patient. How many readings can be stored in the memory?
- \_\_\_\_\_ Discuss how to enter Height, Weight, Respirations or Pain Level into the CVSM 6000 device.
- \_\_\_\_\_ True or False: Cavi-Wipes™, Sani-Cloth® Plus, 10% chlorine bleach solution, and 70% alcohol are all approved cleaning solutions for the device.
- \_\_\_\_\_ Demonstrate how to properly turn off the CVSM 6000 device.