

Clinician Notifier Competency Test

Name (print): _____ Unit: _____ Date: _____

1. When your mobile device has received notification of one or more Acuity® System patient alarms, and one or more are not suspended, the alarm notification bar will be which color?
 - a. Red
 - b. Orange
 - c. Green
 - d. Purple

2. True or False: The Patient List screen displays patients with confirmed and unconfirmed ID numbers, currently monitored at Acuity.

3. Escalation Mode refers to the delivery of alarms to clinicians, in which order:
 - a. Primary Notification only
 - b. Secondary Notification only
 - c. Broadcast Notification only
 - d. Primary, Secondary, and then Broadcast Notification if no one has suspended the alarm.

4. True or False: If a patient is in an alarm state and **Suspend** is selected, all alarms to the device are suspended for 90 seconds.

5. A patient alarm (other than a lethal arrhythmia) is labeled under Active Alarms as a:
 - a. Red Bell
 - b. Red Bell with a line through it
 - c. Yellow Bell
 - d. Green Bell

6. When viewing the Realtime waveform for a patient, how many seconds of data can be visualized on the screen?
 - a. 9 seconds
 - b. 6 seconds
 - c. 7 seconds
 - d. 3 seconds

7. Which of the following alarms will provide alarm notification at the device?
 - a. SpO2
 - b. Arrhythmias
 - c. NIBP
 - d. All of the above

8. True or False: The term “lethal arrhythmia” refers to Asystole, VFIB, and PVC Run.

9. True or False: Patient alarm and equipment alert windows pop up in the foreground of your device screen, even if you are viewing another program.

10. True or False: Tap **ok** to *log out*, and tap it again to *exit* Clinician Notifier.

Clinician Notifier Competency Test Answer Sheet

Name: _____

Unit: _____

Date: _____

1. _____

2. _____

3. _____

4. _____

5. _____

6. _____

7. _____

8. _____

9. _____

10. _____

Clinician Notifier Competency Test

Answer Key

1. a
2. True
3. d
4. True
5. c
6. c
7. d
8. False: Lethal refers to asystole, v-fib and v-tach
9. True
10. True