

Connex® ProBP™ 3400 Digital Blood Pressure Device Competency Test

Name (print): _____ Unit: _____ Date: _____

1. The ProBP 3400 can measure systolic and diastolic blood pressure readings for which patients:
 - a. Adults
 - b. Pediatrics (3 years of age and older)
 - c. Neonates
 - d. Both a and b
2. True or False: The ProBP 3400 operates on one lithium-ion rechargeable battery.
3. The battery for the ProBP 3400 is fully charged if the battery indicator is what color?
 - a. Amber
 - b. Red
 - c. Green
 - d. Blue
4. Which of the following is true when obtaining a blood pressure reading?
 - a. Select the appropriate size cuff
 - b. Position the cuff on bare skin with artery index marker over the artery
 - c. Wrap the cuff so that no more than one or two fingers fit between the cuff and patient's skin
 - d. During the BP measurement, the patient should remain quiet and not move the extremity
 - e. All of the above
5. The following may interfere with accurate blood pressure readings:
 - a. Incorrect cuff size
 - b. Patient's arm position
 - c. Blood pressure taken over clothing
 - d. Arrhythmia
 - e. All of the above
6. (If Applicable) True or False: The Sure BP Blood Pressure will ONLY take a blood pressure upon inflation.
7. True or False: You can configure the ProBP 3400 to take automatic NIBP measurements at variable intervals.
8. True or False: To review **saved readings** from the Home screen, press the **Left Navigation** button. The

review menu option is highlighted. Press the **Select** button. Use **Up** or **Down. Navigation** buttons to review the desired reading.

9. How many blood pressure readings are retained in the ProBP 3400 memory?
 - a. 10
 - b. 25
 - c. 50
 - d. 100

10. Which of the following solutions can be used when cleaning the ProBP 3400?
 - a. Cavi-Wipes™ or Sani-Cloth® Plus
 - b. 70% isopropyl alcohol
 - c. 10% chlorine bleach/90% water solution
 - d. Any of the above

ProBP 3400 Competency Test

Answer Sheet

Name: _____

Unit: _____

Date: _____

1. _____

2. _____

3. _____

4. _____

5. _____

6. _____

7. _____

8. _____

9. _____

10. _____

ProBP 3400 Competency Test

Answer Key

1. d
2. True
3. c
4. e
5. e
6. False
7. False
8. True
9. c
10. d