

Propaq CS Competency Test

1. How do you determine that the Propaq CS battery is being charged?
 - a. Green light on the right side of monitor is illuminated
 - b. Amber light in upper right corner of monitor is illuminated
 - c. Black cable on right of monitor is attached
 - d. a and c

2. On the Propaq CS, which buttons do you press to change patient modes (adult, pediatric, neonatal)?
 - a. SETUP → MORE → WAVE SEL
 - b. SETUP → MORE → CHANGE (select mode) → press YES to confirm
 - c. SETUP → ACUITY → NET OFF
 - d. none of the above

3. What does the house icon represent on the bottom right hand corner of the monitor?
 - a. Home-Return to Main Menu
 - b. Set up menu
 - c. Alarm suspend button
 - d. Obtain a Stat BP

4. How do you silence an alarm at the bedside monitor?
 - a. Turn off the monitor
 - b. Press the icon with a bell in the upper right corner
 - c. Press the icon that resembles a house in the lower right corner
 - d. a and b

5. How long will an alarm remain silenced?
 - a. 30 seconds
 - b. 60 seconds
 - c. 90 seconds
 - d. 120 seconds

6. True or False?

You can resume an alarm after you have silenced it without having to wait for the system to reset itself.

7. To obtain a non-invasive blood pressure:
 - a. Press Setup, Alarms, Limits
 - b. Press the icon that has a blood pressure cuff and an arm
 - c. Press NIBP, Start
 - d. b and c

8. To initiate an automatic NIBP and set the time interval:
 - a. Press NIBP and press interval

- b. Press ECG/RESP, then MORE
 - c. Press NIBP, Press AUTO/MANUAL, and Press Interval to select time
 - d. All of the above
9. To change ECG leads displayed on the monitor screen:
- a. Press ECG/RESP, then press ECG1/ECG2 and select the desired vector
 - b. Press ECG/RESP, then MORE
 - c. Press SETUP, WAVE SELECT, then MORE
 - d. a and c
10. Which leads are available to monitor respirations?
- a. LD 1 (RA – LA).
 - b. LD 2 (RA – LL).
 - c. a and b
 - d. Respiration can not be monitored
11. To change alarm limits for each monitored vital sign:
- a. Press SETUP, MORE, WAVE SELECT
 - b. Press ECG/RESP, MORE, MORE
 - c. Press SETUP, TRENDS
 - d. Press SETUP, ALARMS, LIMITS then highlight parameter to adjust
12. To temporarily disconnect a patient from the Propaq CS, you should:
- a. Turn the monitor off and remove sensors from patient
 - b. Remove sensors from patient and acknowledge the equipment alert
 - c. Remove sensors from the patient
 - b. Remove sensors from the patient and press the alarm suspend button
13. What is OxyCRG?
- a. It graphically compares HR/PR and SpO₂ trends
 - b. It graphically compares CO₂ and respiratory waveforms
 - c. It enables you to view oxygen cardiorespirogram information
 - d. all of the above
14. You should turn the Propaq CS monitor off in between patients:
- a. Sometimes
 - b. Never
 - c. For temporary disconnects only
 - d. Always

Propaq CS Competency Test

Answer Sheet

Name: _____

Unit: _____

Date: _____

1. _____

2. _____

3. _____

4. _____

5. _____

6. _____

7. _____

8. _____

9. _____

10. _____

11. _____

12. _____

13. _____

14. _____

Propaq CS Competency Test

Answer Key

1. d

2. b
3. a
4. b
5. c
6. True
7. d
8. c
9. a
10. c
11. d
12. b
13. d
14. d