

# Propaq LT Competency Checklist

Name (print): \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

- \_\_\_\_\_ Remove the Propaq LT from the charging cradle. True or False: The Propaq LT should be stored in the cradle when not in ambulatory use.
- \_\_\_\_\_ Explain how to appropriately attach leads to a patient and cables to the monitor. Turn on the Propaq LT.
- \_\_\_\_\_ Determine patient mode (adult, pediatric, neonatal). Demonstrate how to change the modes.
- \_\_\_\_\_ Locate the wireless symbol **Y**. Is the Propaq LT connected to the Acuity Workstation?
- \_\_\_\_\_ Locate the Display button and scroll through the different display views.
- \_\_\_\_\_ Obtain a manual NIBP. Change the NIBP to automatic mode; now change the time intervals.
- \_\_\_\_\_ Silence an audible alarm. How long will it remain silenced? a) 30 sec b) 60 sec c) 90 sec
- \_\_\_\_\_ Demonstrate how to change alarm limits for each monitored vital sign.
- \_\_\_\_\_ Obtain a Snapshot of your patient waveform. How many snapshots can be stored in the monitor?
- \_\_\_\_\_ Display patient vital sign trends; how often is data stored in the Propaq LT?
- \_\_\_\_\_ Demonstrate how to properly discontinue monitoring from the Propaq LT (Temporary and Permanent)