

# SureTemp® Plus Competency Checklist

Name (print): \_\_\_\_\_ Unit: \_\_\_\_\_ Date: \_\_\_\_\_

\_\_\_\_ Demonstrate selection and installation of correct probe well and probe for an Oral temperature  
(Hint: Red or Blue)

\_\_\_\_ True or False: Adult Axillary Mode is used for patients 18 years and older.

\_\_\_\_ Demonstrate proper probe positioning for oral measurements, then take a reading

\_\_\_\_ Recall the last temperature taken; what was the reading?

\_\_\_\_ Demonstrate how to switch between Celsius and Fahrenheit

\_\_\_\_ (If applicable) Access the pulse timer

\_\_\_\_ Demonstrate proper probe placement for axillary measurements

\_\_\_\_ Pediatric Axillary Mode is used on patients: a) 14 yrs. & younger b) 15 yrs. & younger  
c) 16 yrs. & younger d) 17 yrs. & younger

\_\_\_\_ Verbalize two reasons why Monitor Mode automatically switches on

\_\_\_\_ True or False: In Monitor Mode, the thermometer does not beep to indicate a final temperature reading.