

Change patient mode _____ MAIN MENU, SETUP, MORE, CHANGE

Change ECG lead _____ MAIN MENU, ECG, ECG LEAD

Change RESP lead _____ MAIN MENU, ECG/RESP, MORE,
(NEXT, CHANGE)

Specify auto or manual NIBP _____ MAIN MENU, NIBP, AUTO/MAN

Specify waveforms for display _____ MAIN MENU, SETUP, WAVE SEL, (NEXT, ON/OFF)



Change HR/PR tone volume _____ MAIN MENU, SETUP, MORE, (NEXT, CHANGE)

View vital sign trends _____ MAIN MENU, SETUP, TRENDS, (NXT TRND)

Change time/date _____ MAIN MENU, SETUP, MORE, MORE,
(NEXT, UP or DOWN)



Alarms and Alerts

Silence alarm or alert tone for 90 sec
or until a new alarm or alert occurs _____   or SILENCE

Inhibit alarm and alert tones for 4 min _____ MAIN MENU, SETUP, ALARMS, 4 SUSPND


Re-enable a silenced alarm or alert tone _____   or RESUME

Change alarm tone volume _____ MAIN MENU, SETUP, MORE, (NEXT, CHANGE)

Change alarm limits _____ MAIN MENU, SETUP, ALARMS, LIMITS



Propaq® Encore Vital Signs Monitor

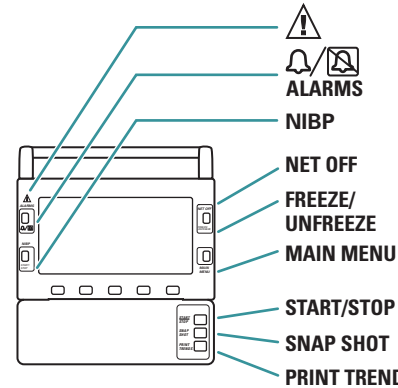

▪ Consult Directions for Use or Reference Guide

▪ Silence/Re-enable a patient alarm or equipment alert

▪ Start/Stop an NIBP reading

▪ Disconnect from Acuity System (if connected)

▪ Freeze/Unfreeze displayed waveforms (if not connected to Acuity)

▪ Return to main menu

▪ Start/Stop printing

▪ Print previous eight seconds

▪ Print specified trends

©2005 · Printed in USA · P/N: 810-0665-03 Rev. A/6/05

800 289 2500 • 503 530 7500 • Beaverton, Oregon USA

WelchAllyn®